


Connecting the LIFEPAK 15 to TabletPCR

On the LIFEPAK 15

- Turn the speed dial and select the Bluetooth icon  along the top
- Click on **Connect**
- Click on **Find Devices**
- Once **MEDICXX-TAB** shows up, click **Stop**
- Turn the speed dial and select the **MEDICXX-TAB**


On the tablet

- Tap the popup in the bottom right saying **Add a Device, Tap to setup your LP15-XX**
- Enter **0000** on the **Pair device?** popup and tap **Allow**
- Both devices should have a Connection Successful message
- The Bluetooth icon on the LIFEPAK's screen will also have a light illuminated

To transmit data to TabletPCR

- Access either Transmit or Archives
- Confirm the **Site** is set to **ZOLL**
- Within TabletPCR, go to **Vital Signs** and **Medical Devices**
- Click **Import**
- Select Physio-Control LIFEPAK 12/15

Troubleshooting the connection

- Confirm on the tablet that the Bluetooth icon is showing on the taskbar. If not:
 - Click the Windows button
 - Type in **Bluetooth and other devices settings**
 - Confirm Bluetooth is On
 - Scroll down to **Related settings** and choose **More Bluetooth options**
 - Confirm **Allow Bluetooth device to find this PC** is checked
 - Click the **COM Ports** tab
 - Confirm **COM5 – Incoming** shows in the list
 - If not, click the **Add** button then select **OK** on the next screen
 - If any COM ports besides **COM5** are added, please select them and click **remove**
- Confirm the **LIFENET Device Communications**  icon is showing in the task bar. If not:
 - Click the Windows button
 - Type **Lifenet Device Communications**
- Confirm **TabletPCR** has the correct **PORT**
 - Select **Vital Signs > Medical Devices > Physio-Control LIFEPAK 12/15**
 - Select **Options**
 - Confirm **Upload Method** is **BlueTooth**
 - Confirm **Port** is **COM 5**